

NiTiShaper

Dispositif médical de classe IIa

Avant d'utiliser NiTiShaper

- 1 - Créer une cavité d'accès
- 2 - Explorer le canal avec une lime K 10, 12 ou 15



Irrigation

①

T1 : sécurisation
4 % / L : 25 mm



Exploration Lime K
+ irrigation

②

T2 : mise en forme
6 % / L : 25 mm



Exploration Lime K
+ irrigation

③

T3 : finition
6 % / L : 25 mm



Vitesse de rotation : 300 à 500 tr/min
Couple : 3 N.cm / 4 N.cm

Légère pression, combinée
à des mouvements de brossage



CE
0459

FFDM-PNEUMAT

78/80, Av. de la Prospective - 18020 Bourges Cedex - France

WWW.THOMAS-DENTALTOOLS.COM

Endo since 1946

REV0-07/2014

NiTiShaper

Medical device class IIa

Before to use **NiTiShaper**

- 1 - Create an access cavity
- 2 - Explore the root canal by using a manual K file 10, 12 or 15



Irrigation

①

T1 : secure of the root canal
4 % / L : 25 mm



Exploration with K File
+ irrigation

②

T2 : shaping
6 % / L : 25 mm



Exploration with K File
+ irrigation

③

T3 : finishing
6 % / L : 25 mm



Rotation speed : 300 to 500 rev/min
Torque : 3 N.cm / 4 N.cm

Slight pressure combined
to brushing movements



CE
0459

FFDM-PNEUMAT

78/80, Av. de la Prospective - 18020 Bourges Cedex - France

WWW.THOMAS-DENTALTOOLS.COM

Endo since 1946

REV0-07/2014